

- **Permobil ESP Configuration for Switched Input Device**

When using a CoMoveIT Smart with a Permobil wheelchair, please ensure that the speeds on each driving profile are set under **ESP** tab, **Speeds** section. Additionally, due to the fact that CoMoveIT Smart is digital (switched input) device, ensure that you set the **Switched Input** to “**Yes**”.

ESP		CoMoveIT Smart IN
Speeds		
<input type="checkbox"/> Maximum Forward Speed	80 %	
<input type="checkbox"/> Minimum Forward Speed	9 %	
<input type="checkbox"/> Maximum Reverse Speed	35 %	
<input type="checkbox"/> Minimum Reverse Speed	7 %	
<input type="checkbox"/> Maximum Turning Speed	50 %	
<input type="checkbox"/> Minimum Turning Speed	23 %	
<input type="checkbox"/> Maximum Forward Acceleration	20 %	
<input type="checkbox"/> Minimum Forward Acceleration	10 %	
<input type="checkbox"/> Maximum Forward Deceleration	20 %	
<input type="checkbox"/> Minimum Forward Deceleration	10 %	
<input type="checkbox"/> Maximum Reverse Acceleration	15 %	
<input type="checkbox"/> Minimum Reverse Acceleration	10 %	
<input type="checkbox"/> Maximum Reverse Deceleration	15 %	
<input type="checkbox"/> Minimum Reverse Deceleration	8 %	
<input type="checkbox"/> Maximum Turn Acceleration	30 %	
<input type="checkbox"/> Minimum Turn Acceleration	20 %	
<input type="checkbox"/> Maximum Turn Deceleration	35 %	
<input type="checkbox"/> Minimum Turn Deceleration	25 %	
<input type="checkbox"/> Fast Brake Rate	25 %	
<input type="checkbox"/> Switched Input	Yes	